

Your orthodontic treatment plan

clearcorrect
A Straumann Group Brand

PATIENT NAME

LOGO HERE - DRAG & DROP IMAGE



PRACTICE NAME

PRACTICE ADDRESS

PRACTICE WEBSITE

PRACTICE NUMBER

Your Proposed ClearCorrect Treatment

This document outlines our proposed plan of treatment which we will discuss with you in more detail.

ClearCorrect aligners are custom-made just for you. With ClearCorrect, you can live life on your terms while discreetly and comfortably straightening your teeth.

ClearCorrect aligners are removeable, so you can eat, drink, and take care of your teeth as you normally would. In fact, you can wear them or remove them as you see fit during any of your daily activities with no added risk of injury or irritation.

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
ClearCorrect aligners are also nearly invisible, so most people won't even know you're wearing them.

After you're happy with the plan we're presenting to you, ClearCorrect will manufacture your aligners. From there you will simply need to wear the aligners 22 hours a day. They'll move your teeth little by little with each aligner.



We look forward to helping you on your journey to getting the smile you deserve!


ABOUT US



YOUR PHOTOS



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YOUR TREATMENT



[VIEW YOUR 3D SETUP](#)

PASSWORD TO ACCESS THE TREATMENT SET-UP:

YOUR INVESTMENT

Total cost of treatment

Payment options

OPTION 1:

OPTION 2:

OPTION 3:

Testimonials

★★★★★

“My smile looks fantastic now and nobody even knows that I was having treatment. It really is invisible.”

JAMES T

★★★★★

“ClearCorrect treatment was so easy and I loved the fact that I could carry on cleaning my teeth like normal and I could take them out for certain work events. I have lots more confidence now.”

SARAH S

★★★★★

“I feel so much more confident now with my new straight teeth. I’m ready for every photo opportunity and no longer cover my mouth with my hand!”

JUSTINE B

SMILE GALLERY



7373 - BEFORE - CASE COURTESY OF DR. EREN KORUNMUS OF TURKEY.



7373 - AFTER - CASE COURTESY OF DR. EREN KORUNMUS OF TURKEY.



7643 - BEFORE - CASE COURTESY OF DR. ABRAR YOUNUS A OF INDIA



7643 - AFTER - CASE COURTESY OF DR. ABRAR YOUNUS A OF INDIA



7775 - BEFORE - CASE COURTESY OF DR. ALLESANDRO BELLATO OF BRAZIL



7775 - AFTER - CASE COURTESY OF DR. ALLESANDRO BELLATO OF BRAZIL



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
8780 - BEFORE - CASE COURTESY OF DR. YAMAN GURAY OF TURKEY



8780 - AFTER - CASE COURTESY OF DR. YAMAN GURAY OF TURKEY

SEE MORE BEFORE & AFTER TRANSFORMATIONS [HERE](#)

Your Next Steps



FAQs

Successful orthodontic treatment is a partnership between the dentist and the patient. We are dedicated to achieving the best possible result for each patient. If you wear your aligners as instructed, attend all of your appointments, and care for your aligners then your treatment should go as planned.

Is treatment comfortable?

Compared to wires and brackets, clear aligners are proven to be more comfortable.¹ ClearCorrect also improves comfort by applying less initial pressure than other aligners on the market.²

ClearCorrect's advanced technology provides a more custom and comfortable fit for a variety of dental situations.

You may notice changes to your bite - the way your upper and lower teeth come together. While this isn't painful, some people say it does take a little getting used to. Changes to your bite are often the first sign that your ClearCorrect aligners are beginning to work.

The longer you wear your aligners, the more comfortable and natural they'll feel.

Are they really invisible?

With ClearCorrect you can show off your smile and not your treatment. ClearCorrect aligners are nearly invisible, so most people won't even know you're wearing them. They also stay clear: studies show that ClearCorrect aligners resist stains better than other brands.

Will it affect my speech?

You may speak with a slight lisp while your tongue adjusts to the aligners – this usually goes away within a few days.

How do I clean my aligners?

Always remove your aligners to eat. Rinse your aligners with cool water and store them in a safe place. It's important that you brush and floss after every meal to prevent food and drink from getting trapped under the aligners. Clean your aligners with a toothbrush and cool water. For more thorough cleaning, you can use non-abrasive toothpaste, but don't use denture cleaners or mouthwashes. Take extra care to keep your aligners away from alcohol and hot, sweet, or colored liquids.

What are those weird bumps on my teeth?

These bumps (called engagers) help the aligner to move your teeth. The great news is, ClearCorrect aligners are specifically designed to effectively move teeth with fewer engagers.

How long does treatment take?

Treatment time can vary based on a variety of factors. Some patients can wrap up treatment in only a few weeks, while others may take longer. You'll be able to see the expected number of steps in your treatment plan, and calculate the time required. Depending on your specific case, you may need to wear each individual step anywhere from 1-3 weeks each.

What is IPR?

IPR is short for 'interproximal reduction'. It's a technique that involves carefully removing small amounts of enamel from between your teeth to reduce their width. IPR is sometimes necessary during treatment to create the space needed to straighten crowded teeth.

What is the daily wear time?

You should wear each aligner all day and night except while eating, drinking, and brushing your teeth. It's important that you wear your aligners at least 22 hours a day, or the straightening process can be slowed which prolongs your treatment.

Will I need to wear a retainer?

When your treatment comes to an end we'll work with you on a retention plan that is best suited for you. This may include a ClearCorrect retainer, a wired (hawley) retainer, or a permanent retainer. Without a retention solution in place, your teeth can begin to shift after your treatment is complete.

What are ClearCorrect aligners made of?

ClearCorrect aligners are made from ClearQuartz, an industry-leading tri-layer plastic material that improves comfort and clarity while maintaining durability.

Do the aligners contain BPA or phthalates?

No. ClearCorrect aligners are made without BPA or phthalates.

Can I still wear ClearCorrect if I'm pregnant or nursing?

Yes! ClearCorrect aligners are free from harmful chemicals and suitable for use when pregnant or breastfeeding. Always ask a dental professional if you are unsure or have questions.

